


# Printing Instruction


1. Print this file on your computer and select ACTUAL SIZE or NO SCALING. This is a sample print dialog box but your printer may have different options.

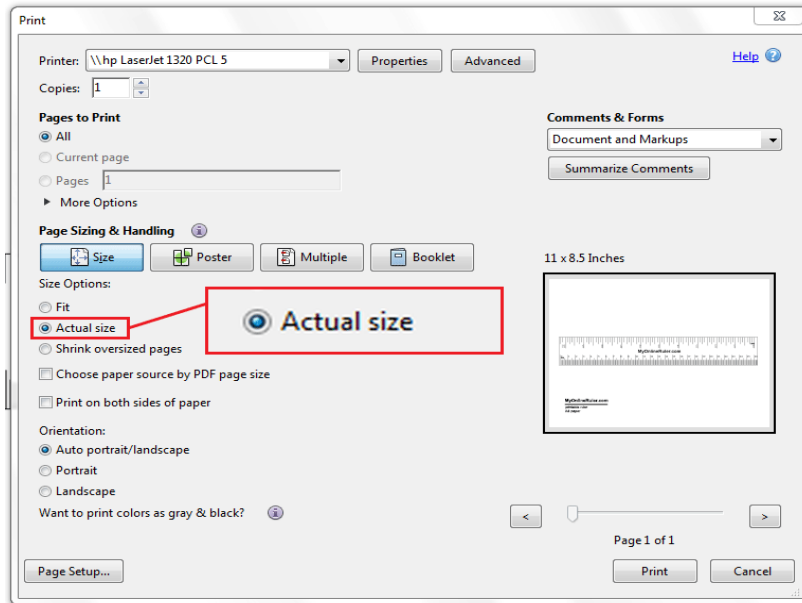

2. Verify that your print is the correct size by placing a CREDIT CARD over this image. The sizes should be identical otherwise you may have resized your print. If so, please review point 1 above.



3. Cut out the Calibration Strip at the bottom of the page
4. Place Calibration Strip on your forehead using clear tape. Attach tape to the center of the strip – not on the ends.
5. Have a friend capture a photo or 'head shot' from about 6' away using the ZOOM control to focus on your head. Look straight ahead, both ears visible.
6. Save and UPLOAD the Measurement Photo to your account.

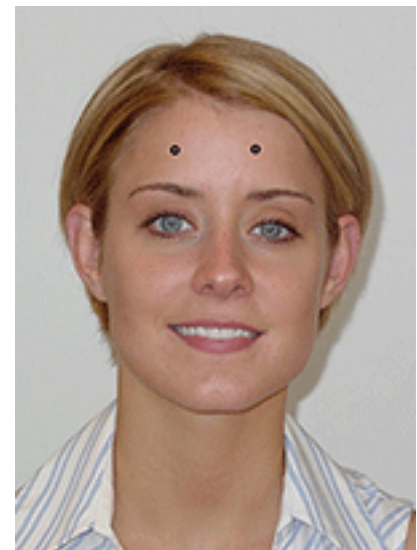

Cut on dotted lines

